



Phenaphthazine Paper

Accutest® Phenaphthazine Paper QC Procedure

Each lot has been verified for accuracy using NIST traceable standards. Expired rolls should be disposed of. Unopened, unexpired rolls do not require further QC testing, nor do rolls that have been opened less than 30 days. Rolls that have been opened more than 30 days may be retested to ensure proper performance by following these procedures using NIST traceable buffers.

Quality Control Procedure

1. Tear off a strip of paper from the roll.
2. Dispense a drop of the pH 5.0 buffer solution onto the strip of pH paper.
3. Immediately match the color of the pH paper with the closest color on the color chart supplied with the paper dispenser.
4. Repeat steps 1-3 with the pH 7.0 buffer solution, or with the buffer required by your testing protocol.

Note:

Color comparison recommended under a combination of fluorescent light and daylight. The color of the base stock may vary from lot to lot, from tan to olive green. This will not affect the accuracy of results.

JANT PHARMACAL
CORPORATION

LABORATORY & POINT OF CARE DIAGNOSTICS

JANT PHARMACAL CORPORATION

16530 VENTURA BLVD, SUITE 512 ENCINO, CA 91436

PHONE: 800.676.5565 818.986.8530 FAX: 818.986.0235

info@jantdx.com www.jantdx.com