## accutest pH Phenaphthazine Paper

## **Accutest® pH brand Nitrazine Indicator Paper**



Accutest® brand nitrazine indicator paper is the economical choice among FDA-listed medical pH test products for the in-vitro determination of body-fluid pH in the 4.5–7.5 range. It is widely used in hospitals and doctors' offices to detect the presence of amniotic fluid in vaginal secretions during late-term pregnancy, a sign of premature amniotic sac rupture. Packaged in a convenient, easy-to-use dispenser complete with color chart, each roll of Accutest® pH Nitrazine Indicator Paper provides approximately 100 tests.

- The economical alternative to higher priced nitrazine papers
- Test papers give instant, easy-to-read results, with clear, bright matches at 0.5 intervals from pH 4.5 to 7.5
- Color charts are lab-tested for precision accuracy
- Made in the U.S.A. with state-of-the-art materials
  - An FDA-listed medical test product, used in hospitals and doctor's offices around the world



## JANT PHARMACAL CORPORATION

16530 Ventura Boulevard, Suite 512, Encino CA 91436 800.676.5565 818.986.8530 Fax 818.986.0235 www.accutest.net info@accutest.net



# accutest pH Phenaphthazine Paper



### Accutest® brand Nitrazine indicator Paper 4.5-7.5

- Each kit consists of a 15 ft. x 1/4 in. roll of premium test paper in a handy dispenser with matching color chart
- Approximately 100 tests per kit

 catalog no.
 pH range
 pH readings

 PF418
 4.5-7.5
 4.5|5.0|5.5|6.0|6.5|7.0|7.5|

## How to Use Accutest® pH Phenaphthazine Paper

- Collect a fresh sample in a clean receptacle.
- Tear off a strip of Accutest® paper and dip into the sample.
- Remove the test paper strip, shake off excess liquid, and compare the strip to the color chart.
- For greatest accuracy, color chart comparisons should be conducted in a combination of fluorescent light and daylight.
- Store Accutest® paper at room temperature, out of direct sunlight.



## JANT PHARMACAL CORPORATION

16530 Ventura Boulevard, Suite 512, Encino CA 91436

800.676.5565 818.986.8530 Fax 818.986.0235

www.accutest.net info@accutest.net

